

## CircuFlow™ 5150

### Quick Setup Guide



**TABLE OF CONTENTS**

<b>Setup</b>	<b>03</b>
<b>Treatment</b>	<b>05</b>
<b>Notes</b>	<b>07</b>

## SETUP

- 01** Open the shipping box and lift device up and out of box.



- 02** Remove the protective foam covers on each side and the plastic bag.



- 03** Check to make sure the box also contains an air blocker and a user manual.



- 04** Place device on a flat and stable surface near the patient.

**\*Not Pictured**

- 05** Unwrap the power cord and plug device into a properly grounded outlet.



## THERAPY SETUP

(Settings can only be modified or restored before or in between treatments.)


**06** Insert air blocker and sleeve (or two sleeves) into the quick connector ports on the front panel of the device.



**07** Press the main power switch down to "ON" which is located on the right side of the device. There will be a single beep to indicate the system is ready to run. The green operation light on the front panel will illuminate.



**08** In order to set the treatment time press down "Mode" button for 5 seconds until there is a beep and the orange "Setting" light illuminates. To change the time, press the arrow buttons to set the time from 30 minutes to 3 hours in 30 minute intervals. To confirm selection, press "Mode" button again and the timer will stop flashing.

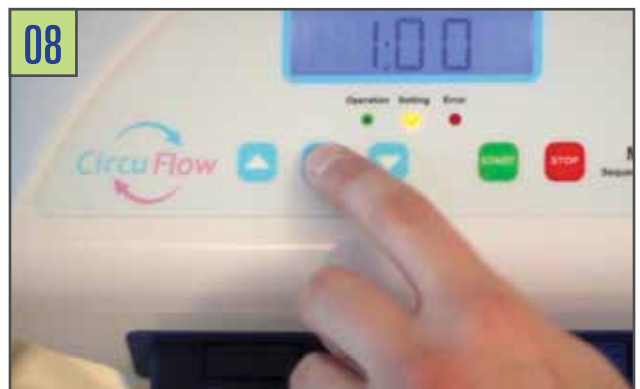

**09** To set the pressure, press "Mode" again and pressure readings will appear. Set pressure as timer was set from 20-80mmHg in 1mmHg intervals. To confirm selection, press "Mode" again. Press "Mode" once more to enter the total use time screen and another to return to the start menu.



**10** Press "START" to turn the pump on time therapy with the selected settings.

**\*Not Pictured**

## TREATMENT

**01** Attach left and/or right quick connectors of the garment to the quick connect ports which are located on the front panel of the device. During a single garment session (i.e. left only or right only) insert the air blocker into the unused quick connect port.



**02** Putting the garment on: a) for leg garments, unzip the garment all the way to the end. Place the foot at the bottom end of the garment and pull up the zipper while supporting the garment to wrap around the leg; b) for arm garments, slide the arm through the internal cavity of the garment.



**03** Press the main power switch down to "ON" which is located on the right side of the device. There will be a single beep and the green power indicator on the front panel will illuminate.

**\*Not Pictured**

**04** Press "START" and the pump will begin to prefill.



**05** The pump will take up to 10 minutes to reach calibrated pressure (larger sleeves may require more time). The garment needs to fill up with air before beginning therapy.



**06** Each treatment will end after its set treatment time has elapsed. The user can also end the treatment at any time during a treatment session by pressing the “STOP” button on the front panel.

**\*Note:** if the user presses “STOP” the treatment will not pick up from where it left off at.



**07** After the treatment, the device will vacuum air from the garment for 3 minutes so that it will facilitate removal of the garment. You will hear a beep when the vacuum is completed.

**\*Note:** If using one of the larger sleeves all the air may not vacuum out within 3 minutes. The vacuum setting can go through another 3 minutes by pressing the “STOP” button again.



**08** Once the sleeve is deflated, press the power switch on the right side up to the “OFF” position and then unplug the power cord. Once the power indicator light is off, it is safe to remove the garment.



**09** The garment should be loose enough by now so that you can unzip the garment and remove.



# NOTES

---



 **DEVON**  
MEDICAL PRODUCTS