

## Cirona™ Deep Vein Thrombosis (DVT) Therapy Garment

Non-Sterile\* Garments. Intended for single patient use and cleaning is not recommended

### 01 Application Instructions

1. Plug the pump into an electrical outlet but DO NOT SWITCH ON YET.
2. Remove the sleeve(s) from packaging.
3. The garments can be used on either foot depending on conditions.
4. Attach the tube set to the pump by connecting the male connectors into the pump connectors until a click is heard. Pump connectors are located on the right side of the pump.
5. Attach the garments to the tube set using the snap lock connectors.
6. Foot Garment: Fully unfold and position the inflatable chamber against the base of the patient's foot with the tube set facing upward. Wrap the tab firmly over top of the foot. Pulling the remaining long tab around the heel and firmly secure to the top of the foot. *(See reverse side for visual instruction.)*
7. Switch on the device. Please refer to the device's Operation Instructions for how to use the system.

### 02 Cautions

- Ensure that there are no blockages in the tubing and that connectors are firmly locked in place.
- Regularly check the system while it is in operation and ensure the garment fits.
- Garment should be removed if the patient experiences pain, numbness, or tingling.
- Never put on or take off the garments when inflated as it may cause damage to the sleeves.

### 03 Intended Use

The Cirona™ is a prescription-only device intended to be used preventatively to increase venous blood flow in patients at risk of deep vein thrombosis due to the associated risk factors for thrombus formation during: trauma, critical care, general medicine, general surgery, as well as neurological, orthopedic, urologic, obstetric conditions and treatments.

### 04 Contraindications

The Cirona™ should NOT be used on patients with the following conditions:

1. Severe atherosclerosis or other ischemic vascular diseases
2. Suspected or known acute deep vein thrombosis
3. Severe congestive cardiac failure
4. Existing pulmonary edema
5. Extreme deformity of the limbs
6. Any local skin or tissue condition in which the garments would interfere:
  - Gangrene
  - Untreated or infected wounds
  - Recent skin graft
  - Dermatitis
7. Known presence of malignancy in the legs
8. Limb infections, including cellulitis, that have not received antibiotic coverage
9. Presence of lymphangiosarcoma

Model	Name	Size (Instep Circumference)
F-15	Small Foot	Up to 13"
F-10	Standard Foot	Up to 16"

IFU37.0009 Rev C 20141111

## Cirona™ Deep Vein Thrombosis (DVT) Therapy Garment

### 01 Step One



Fully unfold and position the inflatable chamber against the base of the patient's foot with the tube set facing upward.

### 02 Step Two



Wrap one tab on the side over the top of the foot as shown.

### 03 Step Three



Wrap the other tab on the side with hook and loop fastener material firmly over the top of the foot.

### 04 Step Four



Wrap the remaining long strap around the heel and firmly secure to the top of the foot.