

Cirona™ Deep Vein Thrombosis (DVT) Therapy Garment

Non-Sterile* Garments. Intended for single patient use and cleaning is not recommended

01 Application Instructions

1. Plug the pump into an electrical outlet but DO NOT SWITCH ON YET.
2. Remove the sleeve(s) from packaging.
3. The garments can be used on either leg depending on conditions.
4. Attach the tube set to the pump by connecting the male connectors into the pump connectors until a click is heard. Pump connectors are located on the right side of the pump.
5. Attach the garments to the tube set using the snap lock connectors.
6. Unfold and position the inflatable chamber directly behind the patient's calf. Make sure that they are snug and the inflatable chambers are in the appropriate position.
7. Switch on the device. Please refer to the device's Operation Instructions for how to use the system.

02 Cautions

- Ensure that there are no blockages in the tubing and that connectors are firmly locked in place.
- Regularly check the system while it is in operation and ensure the garment fits.
- Garment should be removed if the patient experiences pain, numbness, or tingling.
- Never put on or take off the garments when inflated as it may cause damage to the sleeves.

03 Intended Use

The Cirona™ is a prescription-only device intended to be used preventatively to increase venous blood flow in patients at risk of deep vein thrombosis due to the associated risk factors for thrombus formation during: trauma, critical care, general medicine, general surgery, as well as neurological, orthopedic, urologic, obstetric conditions and treatments.

04 Contraindications

The Cirona™ should NOT be used on patients with the following conditions:

1. Severe atherosclerosis or other ischemic vascular diseases
2. Suspected or known acute deep vein thrombosis
3. Severe congestive cardiac failure
4. Existing pulmonary edema
5. Extreme deformity of the limbs
6. Any local skin or tissue condition in which the garments would interfere:
 - Gangrene
 - Untreated or infected wounds
 - Recent skin graft
 - Dermatitis
7. Known presence of malignancy in the legs
8. Limb infections, including cellulitis, that have not received antibiotic coverage
9. Presence of lymphangiosarcoma

Model	Name	Size (Instep Circumference)
D-135	Small Thigh	Up to 22"
D-130	Standard Thigh	Up to 27"
D-140	Large Thigh	Up to 36"
D-145	Extra Large Thigh	Up to 42"

IFU37.0008 Rev C 20141111